


gemini II treatment table

#6004

GEMINI II

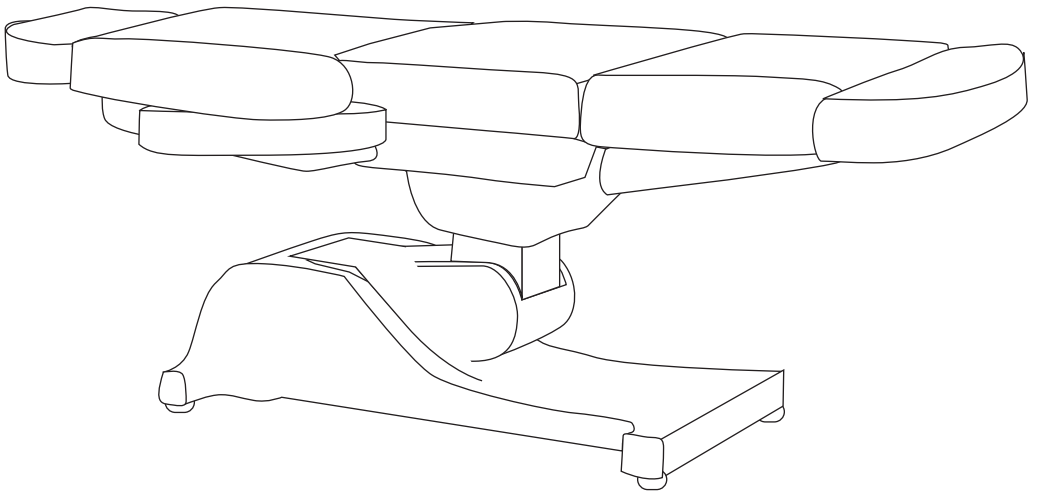



IMPORTANT PLEASE READ THESE SAFETY INSTRUCTIONS

Listed below are essential safety precautions.

GENERAL SAFETY

1. Read carefully and keep these instructions in a safe place.
2. Use this product only for its intended function and do not use attachments not recommended by the manufacturer.
3. Do not use outdoors.
4. Do not try to move the table when in use.

ELECTRICAL APPLIANCES

1. All GPO's must be installed by a qualified Electrician in accordance with local electrical standards.
2. DO NOT allow children to operate the electric motors in this product.
3. Turn the power off and remove the plug when not in use.
4. Turn the power off and remove the plug when cleaning.
5. Do not let the power cord get damaged in any way.
6. Keep the power cord away from heat sources.
7. Do not use an extension cord unless the cord complies with local electrical safety standards.
8. Periodically inspect the power cord for signs of damage.
9. Do not operate any electrical item with a damaged power cord.



IMPORTANT

KEEP FINGERS AND HANDS AWAY FROM TABLE JOINTS UNDER THE TABLE



IMPORTANT

DO NOT ALLOW CHILDREN AROUND THE TREATMENT TABLE WHEN IN USE



IMPORTANT

DO NOT ALLOW CHILDREN TO OPERATE THE ELECTRIC MOTORS THAT MOVE THE TABLE



IMPORTANT

DO NOT USE AROUND WATER OR IN A WET ENVIRONMENT




IMPORTANT

WHEN REPLACING ANY PARTS OR ASSEMBLING THE TABLE, YOU MUST ENSURE THE TABLE IS RESTING ON ITS SIDE TO SAFEGUARD FROM CRUSH INJURY

GET STARTED

TOP VIEW FLAT POSITION

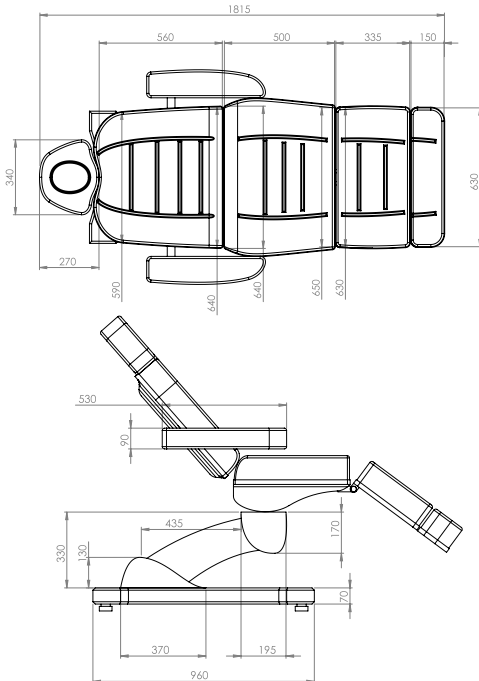

All measurements are in MM.



GEMINI II

Beauty bed with electric adjustment

Width: 650mm (920mm with arms)

Length: 1300mm-1810mm


Height Range: 620mm-1000mm

· Electric height, back, tilt & leg


· Hand held remote control



OPTIONAL UPGRADE BATTERY for back up power supply

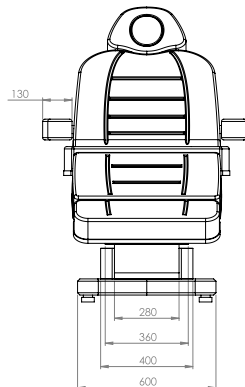

PARTS & ASSEMBLY

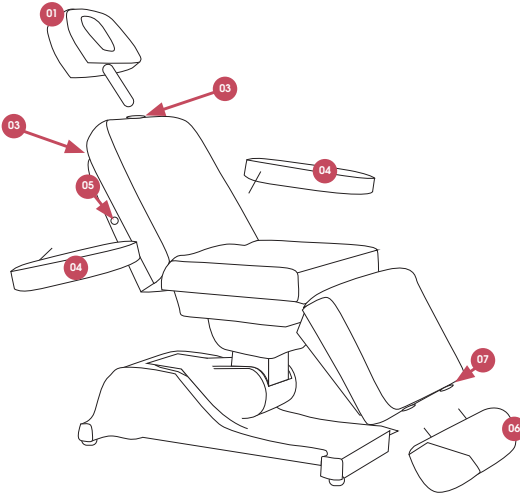


PARTS

1. Removable headrest
2. Headrest Slot
3. Nut (at the rear of the chair)
4. Removable armrest
5. Armrest slot
6. Removable footrest
7. Footrest slot

ASSEMBLY

1. Slot the headrest pole (02) into the slot provided (03) on the top of the chair. Tighten nut at the rear of the chair so headrest is secure.
2. Slot the footrest poles x2 (06) into the slots (07) provided on the end of the chair.
3. Guide each armrest (04) with the pole/bolt into each slot (05) on the left and right of the chair. If the armrest does not lock flat in position, please swap arms over.


OPERATION

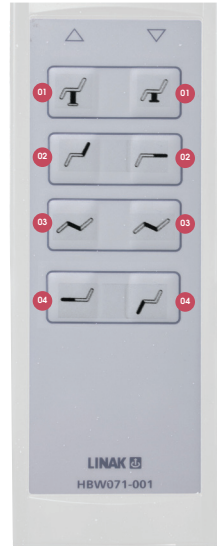

HAND REMOTE CONTROL


1. Table up/down
2. Back Rest up/down
3. Tilt forward/back
4. Leg rest up/down

FUNCTIONS

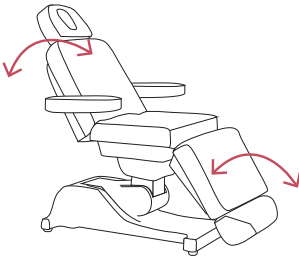


BACK & LEG ADJUSTMENT

Using the hand remote control, adjust the back and leg adjustment by going up or down.

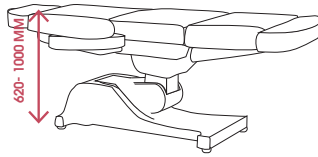

HEIGHT ADJUSTMENT

Using the hand remote control, adjust the table height by going up or down.
Minimum Height: 620mm
Maximum height: 1000mm

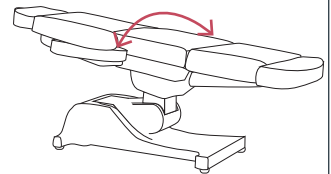

TILT ADJUSTMENT

Using the hand remote control, adjust the tilt of the bed by going forward and back.

WARRANTY PERIOD

PERIOD AND DESCRIPTION	PRODUCTS
12 MONTHS PARTS & LABOR	All products
2 YEARS PARTS ONLY	All products excluding small electrical appliances
3 YEARS PARTS ONLY	All Linak electric actuators
5 YEARS PARTS ONLY	Taps cartridge, Ceramic basins

TERMS AND CONDITIONS

For more information about our terms and conditions including warranty, please refer to our website.

MAINTENANCE

A periodic inspection of all components and fasteners should be made to ensure that your Comfortel Product is structurally intact and functioning properly.

Damaged and broken parts should be replaced and loose fasteners tightened.

All maintenance to be carried out by a suitably qualified tradesperson.

⚠ IMPORTANT
WHEN REPLACING ANY PARTS OR ASSEMBLING THE TABLE, YOU MUST ENSURE THE TABLE IS RESTING ON IT'S SIDE TO SAFEGUARD FROM CRUSH INJURY

CLEANING

UPHOLSTERY

- All upholstery needs some care; good housekeeping will help to prolong the life of any upholstered products.
- Avoid extreme temperatures. Extreme cold can make your vinyl prone to cracking, while extreme heat can cause permanent stretching.
- Avoid exposing to direct sunlight for prolonged periods of time.

ALWAYS FOLLOW THE CLEANING INSTRUCTIONS

- Clean regularly (weekly) to remove body oils which cause hardening
- To clean use a sponge which has been moistened in warm soapy water
- After cleaning with a sponge, dry with a soft, clean cloth
- Stubborn grime may require gentle scrubbing with a soft brush

Do not use:

- acetone
- furniture polishes
- bleach
- detergent
- abrasive cleaners
- steel wool

Damage caused by any improper treatment of this product is not covered by the product warranty.

comfortel

comfortel.com.au

PRODUCT CODE

#6004 GEMINI II TREATMENT TABLE

1 X 10AMP GPO REQUIRED
POWER: 240V, 50-60HZ, MAX 2.4A
MAX LOAD: 150KG

Revision Date: 28.06.2018

©Comfortel Furniture 1997-2018. All rights are reserved.
No part of this document may be photocopied, reproduced or translated to another language without written consent of Comfortel Furniture Pty Ltd.

The information contained in this document is subject to change without notice and should not be construed as a commitment by Comfortel Furniture Pty Ltd.

Comfortel Furniture Pty Ltd. assumes no responsibility for any errors that may appear in this document nor does it make expressed or implied warranty of any kind with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Comfortel Furniture Pty Ltd. shall not be liable for incidental or consequential damages in connection with or arising out of the use of this document and the program material.

PLEASE RETAIN THIS INFORMATION FOR FUTURE REFERENCE

For more information about our products and services, contact us at comfortel.com.au

PURCHASE DATE

INVOICE NUMBER

NOTE YOUR PURCHASE DATE & INVOICE NUMBER FOR FUTURE REFERENCE